


Handling

The Lice Snatcher should be handled with a light hand and a supple wrist. Find a balance where the comb scratches just a little, but so gently that it does not hurt. It is a whole new way to comb hair so it does require some practice, but once you have got the hang of it, it is an extremely effective way to remove head lice from your hair and scalp. (See how to perform the treatment at www.licesnatcher.com).

Before you start

- Assemble the Lice Snatcher as shown on the back, and assemble it onto your vacuum cleaner.
- Find a place with good lighting and a comfortable working height (you may want to place the vacuum cleaner on a table).
- Start by combing the hair thoroughly with a brush or a comb (it must be clean, dry and possibly given a hair conditioner treatment).
- Remove jewellery, hearing aids, glasses and other loose objects before beginning the treatment.

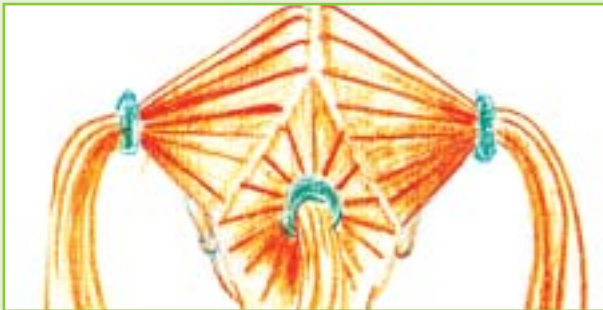


- Part the hair carefully into 3-6 pony-tails by means of hair elastics.
- If your hair is too short for pony-tails, comb the hair systematically from the back of your head and up to the top of your head.

You need good suction

The Lice Snatcher suction must be optimum; if you comb your palm, the Lice Snatcher should "stick" to your palm without problems when you turn your palm downwards. Check the suction regularly during the treatment.

How to perform the treatment:

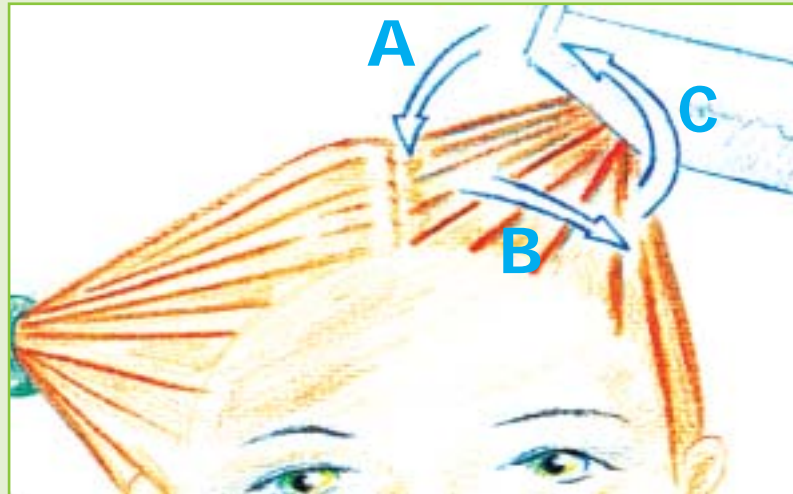

Important

- Do not apply the nozzle to ears and eyes.
- Do not tear at the hair with the small comb if the hair starts matting. Use a brush to comb out the hair instead.

- 1** Suck one pony-tail (after removing the hair elastic) into the nozzle under the comb from the hair ends and up. In order to avoid matting, the hair should hang straight and be able to flutter in the Lice Snatcher nozzle

- 2** Now lead the nozzle all the way up to the parting.

A: Tilt the nozzle downwards to the scalp.

B: Comb the hair from the scalp and approx. 5cm down.


C: Tilt the comb so that it is completely free from the hair.

Repeat this combing procedure ("suction combing") until the scalp has been thoroughly combed. This procedure should take about 4-5 minutes per pony-tail.

- 3** Finish the combing of each hair parting by letting the comb run through the hair from scalp to hair end a couple of times. Use the same technique for combing the next pony-tail. Repeat this procedure until the entire scalp (all the pony-tails) has been thoroughly combed.

After the treatment

Now disassemble the filter house (4 and 6) and look inside the filter. If you have captured any lice, place the lid (5b) on the filter. Repeat the treatment until you no longer capture any lice. Finally, note the number of adult lice and nymphs captured and the date in the Treatment Plan. Discard the sealed filter.

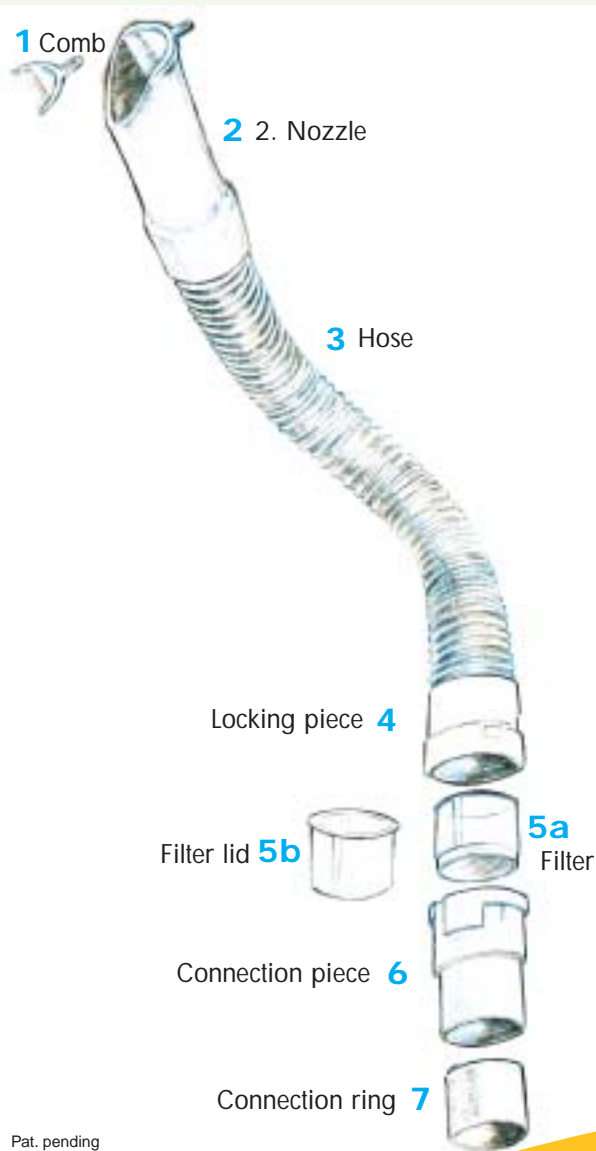

Assembly:

- 1 Assemble the Lice Snatcher (2+3+4) and insert a comb (1).
- 2 Insert a filter (5a) into the connection piece, the bottom of the filter facing the vacuum cleaner.
- 3 Screw the connection piece (6) onto the Lice Snatcher.
- 4 Now assemble the Lice Snatcher onto the vacuum cleaner

The connection ring (7) is only to be used if the vacuum cleaner's connection pipe is a 32mm pipe.

Remember!

The Lice Snatcher only removes the adult head lice - not their eggs. Therefore, the treatment must be continued with another four combing sessions with a 3-4 day interval so that the lice are removed before they become egg-layers. The total treatment period is 14 days and thus follows the louse life cycle. If adult lice (3mm) are later found in connection with a combing session, the lice already captured may have laid eggs that have now hatched. You will have to start all over again with a new treatment!

See how to perform the treatment at www.licesnatcher.com

If you have any queries or problems in connection with the treatment, please contact:

Snatchers Company A/S

Nyvangsvej 5

DK-4160 Herlufmagle

Telephone +45 55 77 41 25

Telefax +45 55 77 45 25

E-mail: info@licesnatcher.com

The Lice Snatcher



Get rid of head lice
- without harming the environment!